



## IGNITE® MIXING INSTRUCTIONS

129596-4

The following languages are included in this packet:

English (en)

For additional languages, visit our website [www.wmt.com](http://www.wmt.com).

Then click on the **Prescribing Information** option.

**For additional information and translations please contact the manufacturer or local distributor.**



CE 0086\*

Wright Medical Technology, Inc.  
5677 Airline Road  
Arlington, TN 38002  
U.S.A.

EC REP

Wright Medical EMEA  
Hoogoorddreef 5  
1101 BA Amsterdam  
The Netherlands

**\* The CE-Marking of Conformity is applied per catalog number and appears on the outer label, if applicable.**

Rx ONLY

September 2012  
Printed in U.S.A.

**IGNITE®  
INJECTABLE STIMULUS FOR  
FRACTURE CALLUS FORMATION  
129596-4**

Instructions For Use  
Mixing Instructions

**SITE PREPARATION\***



**STEP 1**

Localize defect under fluoroscopic control with BLUE 8G needle and trocar.



**STEP 2 (optional)**

If disrupting fibrous tissue within defect is desired, use a longitudinal sweeping motion.



**STEP 3**

Remove trocar and insert IGNITE® Elevator, lining up markers before snapping together. Create a subperiosteal space for graft injection.

**GRAFT PREPARATION**



**STEP 4**

Using the GREEN 11G needle and syringe, aspirate 20cc bone marrow with standard technique. Set aside.



**STEP 5**

Remove BLUE plunger from 30cc syringe. Then in a clockwise motion, twist until completely secure to syringe stand in the blister.



**STEP 6**

Add bone marrow aspirate to 30cc syringe.



**STEP 7**

Carefully add IGNITE® Powder on top of bone marrow aspirate.

**NOTE: Once the IGNITE® powder is combined with the bone marrow aspirate, mixing and injection should be completed within 5 minutes.**



#### STEP 8

Using the provided spatula, mix to an even consistency.

**NOTE:** Make sure no powder clumps remain on bottom.



#### STEP 9

In a clockwise motion, twist and pull syringe barrel AND cap from the blister. Secure syringe cap and carefully insert BLUE plunger into syringe barrel.



#### STEP 10

Loosen syringe cap. Eliminate air by inverting syringe and advancing plunger. Remove syringe cap.

### INJECT GRAFT COMPOSITE



#### STEP 11

Dock graft syringe to pre-placed needle. Inject IGNITE® composite graft into the subperiosteal envelope.

**NOTE:** If injection seems difficult, reposition the needle.



#### STEP 12

After injecting, the needle must remain undisturbed for 3-5 minutes (for partial coagulation). Slowly withdraw needle in small increments to minimize back flow.

\*Proper surgical techniques are necessarily the responsibility of the medical professional. The foregoing guidelines are furnished only as recommended techniques. Each surgeon must evaluate the appropriateness of the techniques based on his or her own medical training and expertise.

™ Trademarks and © Registered marks of Wright Medical Technology, Inc.

Covered by one or more of the following patents. US Patents 5,614,206; 5,807,567; 6,030,636 Patents pending. ©2002 Wright Medical Technology, Inc. All Rights Reserved.