



MOUNTABLE ENDOSCOPIC BLADE

150856-0

The following languages are included in this packet:

English (en)
Español (es)
Türkçe (tk)

Deutsch (de)
Italiano (it)

Nederlands (nl)
Português (pt)

Français (fr)
中文- Chinese (sch)

For additional languages, visit our website www.wmt.com. Then click on the **Prescribing Information** option.

For additional information and translations please contact the manufacturer or local distributor.



Wright Medical Technology, Inc.
1023 Cherry Road
Memphis, TN 38117
U.S.A.

EC REP

Wright Medical UK Ltd
3rd Avenue
Letchworth
Hertfordshire, SG6 2JF
UK

* The CE-Marking of Conformity is applied per catalog number and appears on the outer label, if applicable.

ONLY

October 2013
Printed in U.S.A.

Attention Operating Surgeon
**IMPORTANT MEDICAL
 INFORMATION**
**MOUNTABLE ENDOSCOPIC BLADE
 (150856-0)**

OUTLINE:

DEFINITIONS













GENERAL PRODUCT INFORMATION

- A. DESCRIPTION
- B. INDICATIONS
- C. WARNINGS AND PRECAUTIONS
- D. HANDLING

DEFINITIONS

Symbols and abbreviations may be used on the package label. The following table provides the definition of these symbols and abbreviations.

Table 1. Definitions of Symbols and Abbreviations

Symbol	Definition
	Batch code
	Catalog number
	Do not re-use
	Caution, consult accompanying documents
	Consult operating instructions
	Use by
	Temperature Limitation
	Keep dry
	Keep away from sunlight
	Date of manufacture
	Manufacturer
	Authorized EC Representative in the European Community

STERILE R	Sterilized using radiation
R ONLY	For prescription use only
SS	Stainless Steel

GENERAL PRODUCT INFORMATION

A. DESCRIPTION

The Mountable Endoscopic Blade consists of sterile, single-use, disposable saw blades. The blades should be used only with a 4mm 30° endoscope.

B. INDICATIONS

The Mountable Endoscopic Blade is indicated for use in minimally invasive ligament or fascia release.

- Plantar fasciotomy for plantar fasciitis
- Morton's neuroma for interdigital neuroma
- Gastrocnemius release for heel cord contracture (equinus)

C. WARNINGS AND PRECAUTIONS

Proper surgical techniques are the responsibility of the medical professional. The Mountable Endoscopic Blade is furnished as a tool to facilitate minimally invasive ligament or fascia release. Each surgeon must evaluate the appropriateness of the instruments and techniques for each patient based on his or her own medical training and expertise. As with any surgical procedure, care should be exercised in treating individuals with preexisting conditions that may affect the success of the surgical procedure. Every patient is different and patient results may vary.

Devices labeled for single-use only should never be reused. Reuse of these devices may potentially result in serious patient harm. Examples of hazards related to the reuse of these devices include, but are not limited to: significant degradation in device performance, cross-infection, and contamination.

D. HANDLING

The Mountable Endoscopic Blade is provided sterile and should be considered sterile unless the inner packaging has been opened or damaged. This product should not be resterilized. The Mountable Endoscopic Blade is for single patient use and should never be reused. Used or expired product should be properly discarded. The Mountable Endoscopic Blade should only be used with a 4mm 30° endoscope.