



# **ALLOMATRIX<sup>®</sup> Injectable Putty Mixing Instructions**

**146618-1**

The following languages are included in this packet:

English (en)

For additional information and translations please contact the manufacturer or local distributor.



Wright Medical Technology, Inc.  
5677 Airline Rd.  
Arlington, TN 38002  
U.S.A

**R<sub>x</sub> ONLY**  
June 2011  
Printed in U.S.A.

# ALLOMATRIX<sup>®</sup> Injectable Putty

## Mixing Instructions

(146618-1)

**NOTE:** Do not add any substances to the putty that are not recommended within these mixing instructions. Using alternative mixing solutions and/or adding other substances to the mixture may alter the safety and effectiveness of this product.



**1)** Using sterile technique with gloves, empty powder into the mixing bowl.



**2)** Empty mixing solution into the bowl.



**3)** Mix with spatula and knead material against the side wall of the bowl until the desired consistency is achieved (approximately 30-60 seconds).



**4)** After achieving a putty-like consistency, the material can be handled digitally. Material maintains handling characteristics up to 10 minutes after mixing.

*See package insert for additional information.*

ALLOMATRIX® is a registered trademark of Wright Medical Technology, Inc.