PRODUCT INSERT INSTRUMENTS AND SIZERS

19-5002 rev L

INSTRUMENTS AND SIZERS

APPLICATION

Specialty instrumentation and sizers are designed to aid the surgeon in installation, assembly, and/or removal of a surgical device. The surgical techniques for implantation of the device describe the proper application of specialty instrumentation and should be read and understood by the surgeon prior to use.

WARNINGS

- Clean, not sterile, instruments and sizers must be sterilized prior to each use.
- Sizers are designed to aid in implant site preparation only. They must never be used as implants.
- The most common cause of instrument breakage is misuse. Specialty instrumentation should never be used for tasks it was not specifically designed to perform. Misuse of an instrument may result not only in damage to the instrument, but also trauma to the patient and operating room personnel.
- If not handled properly, instruments with cutting edges or sharp corners may compromise sterility by tearing surgical gloves.
- If an instrument tip becomes bent, chipped, or otherwise damaged the instrument should be replaced or repaired before further use. Attempts to straighten bends are not advised as the metallurgical integrity of the metal may be compromised in the process, and the instrument may subsequently break during use.

LIMITED WARRANTY

Tornier, Inc. warrants that this product meets the manufacturer's specifications and is free from manufacturing defects at the time of delivery. This warranty specifically excludes defects resulting from misuse, abuse, or improper handling of the product subsequent to receipt by the purchaser. Tornier, Inc. does not warrant the outcome of the surgical procedure.

CLEANING INSTRUCTIONS

- New instruments are packaged clean, not sterile and should be assumed to be contaminated. Instruments must be cleaned and sterilized before each use.
- For your safety, be familiar with the procedures for handling contaminated materials at your facility before following these instructions.
- Clean instruments as soon as possible after use and avoid allowing soiled instruments to dry. Immerse, use damp towels or sponges with deionized or distilled water to keep soiled instruments moist prior to cleaning.
- Manually scrub with a clean, soft-bristled brush or soft cloth in mild detergent following the detergent manufacturer's instructions for use. Avoid using extreme detergent concentration levels. Detergents with enzyme cleaners or separate enzyme solutions may be used, but their effectiveness has not been evaluated. Warm or hot water (300°F/149°C maximum) should be used to aid in cleaning. Ultrasonic cleaning is recommended for instruments with internal surfaces or crevices which may be hard to clean manually.
- pH neutral cleaners (pH 6.0-8.0) are recommended for longer life of the instruments. If acidic or alkaline solutions are used follow the manufacturer's recommendations for neutralizing the pH by rinsing with water or neutralizing agent. Highly alkaline or acidic cleaners (used in some mechanical washers) are not recommended as they will reduce the life of the instruments and may affect instrument performance. Avoid prolonged exposure to acidic or alkaline solutions and solutions containing chlorides, bromides or iodine.

- Use of water soluble lubricants is recommended for instruments with moving parts or those intended to be assembled intraoperatively to another instrument.
- Rinse thoroughly with deionized water or distilled water.
- Dry completely before sterilizing.
- Inspect each instrument thoroughly for cleanliness, especially in broach teeth, internal surfaces and crevices. Instruments should be re-washed if there are any signs of foreign matter or residue. Check instruments thoroughly for damage. Do not use damaged instruments as they may compromise the surgical outcome, replace damaged instruments before next use.
- Silicone elastomer sizers may develop a static charge which attracts particulate matter. Sizers may be kept in sterile saline solution to minimize particulate pickup.
- Use of mechanical washers have not been evaluated by the manufacturer. Qualification of specific wash cycles and equipment should be completed by the user.

STERILIZATION

The recommended sterilization method is steam sterilization. EtO sterilization is not recommended for silicone sizers due to the lengthy aeration time required for this material. The following sterilization cycles have been shown to produce a sterility assurance level of 10⁻⁶ when parts have been cleaned to the instructions above. Other similar steam cycles and cleaning procedures may be used but have not been evaluated. Sterilization qualification was performed using specific equipment and procedures. Use of cycles, equipment and procedures other than those listed should be qualified by the user. Do not exceed 300°F/149°C.

- Wrapped Gravity Steam Sterilization 20 minutes minimum @ 270-275°F (133-135°C)
- Wrapped Prevacuum Steam Sterilization 4 minutes minimum @ 270-275°F (133-135°C)
- Unwrapped Prevacuum Steam Sterilization 3 minutes minimum @ 270-275°F (133-135°C)

For further information on the above cleaning and sterilization recommendations please contact Tornier, Inc.

CAUTION: U.S. Federal Law restricts this device to sale by or on the order of a physician.

	CATALOG NUMBER		DO NOT USE IF PACKAG IS DAMAGED
	BATCH CODE		NON-STERILE
	AUTHORIZED REPRESENTATIVE OF THE EUROPEAN COMMUNITY		CAUTION: FEDERAL LAW (USA) RESTRICTS THIS DEVICE TO SALE BY OR ORDER OF A PHYSICIAN
	CAUTION		CE MARK & IDENTIFICATION NUMBER OF NOTIFIED BODY
	CONSULT INSTRUCTIONS FOR USE		MANUFACTURER
	DO NOT RESTERILISE		USE BY
	DO NOT REUSE		
	NOT MADE WITH NATURAL RUBBER LATEX		

Tornier, Inc.
10801 Nesbitt Avenue South
Bloomington, MN 55437
USA
Tel: 1 952 921 7100
Fax: 1 952 236 4007

Tornier S.A.S
161 rue Lavoisier
38330 Montbonnot Saint Martin
France
Tel: 33 (0) 476 613 500
Fax: 33 (0) 476 613 533

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Pour plus d'informations sur les recommandations de nettoyage et de stérilisation, veuillez contacter Tornier, Inc.