



PRO-DENSE™ BONE GRAFT SUBSTITUTE

MIXING INSTRUCTIONS

154103-0


The following languages are included in this packet:

English (en)	Deutsch (de)	Nederlands (nl)	Français (fr)
Español (es)	Italiano (it)	Português (pt)	Türkçe (tk)

For additional languages, visit our website www.wright.com.

Then click on the **Prescribing Information** option.

For additional information and translations please contact the manufacturer or local distributor.

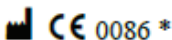

Wright Medical Technology, Inc.
1023 Cherry Road
Memphis, TN 38117
U.S.A.

EC	REP
----	-----

Tornier S.A.S
161 Rue Lavoisier
38330 Montbonnot Saint Martin
France

*** The CE-Marking of Conformity is applied per catalog number and appears on the outer label, if applicable.**

Rx ONLY

July 2018
Printed in U.S.A.

**PRO-DENSE™ BONE GRAFT SUBSTITUTE
(154103-0)**

EN

NOTE: Do not add any additional substances to the graft. Using alternative mixing solutions and/or adding other substances to the mixture may alter the safety, effectiveness and setting of this product.

1. Remove top from mixer and add all powder into bowl.



2. Add all solution to the bowl.



3. Replace mixer top and vigorously mix for 30 secs (approx. 50 revolutions)



Note: Deliver paste within 5 min.

4. Remove top and pack the graft into the desired area using the small spatula provided for graft delivery.



5. (OPTIONAL) Fill the syringe with graft material using the larger spatula. In inverted position, tap on the table to help compact graft.



6. Attach the provided luer cap and cannula.



7. Insert the threaded spindle rod through the back of the syringe. Rotate handle to inject graft.



Note: The graft will set in ~ 20 - 30 min.

Note: Residual material may be left in syringe. The amount of remaining material is relative to user's adherence to the mixing and ejecting times outlined in these instructions.