



## SALVATION™ Midfoot Reconstruction System

153910-1

The following languages are included in this packet:

English (en)

For additional languages, visit our website [www.wright.com](http://www.wright.com). Then click on the **Prescribing Use** option.

**For additional information and translations please contact the manufacturer or local distributor.**



Wright Medical Technology, Inc.  
1023 Cherry Road  
Memphis, TN 38117  
U.S.A.



**Rx ONLY**

December 2018  
Printed in U.S.A.



*Attention Operating Surgeon*  
**IMPORTANT MEDICAL INFORMATION**  
**WRIGHT MEDICAL**  
**SALVATION™ FUSION SYSTEMS**  
**(153910-1)**

OUTLINE:

DEFINITIONS

I. GENERAL PRODUCT INFORMATION

- A. PATIENT SELECTION
- B. CONTRAINDICATIONS
- C. POTENTIAL COMPLICATIONS AND ADVERSE REACTIONS
- D. PRECAUTIONS
- E. MRI SAFETY INFORMATION
- F. HANDLING AND STERILIZATION
- G. STORAGE CONDITIONS












II. SPECIFIC PRODUCT INFORMATION

- A. SALVATION™ MIDFOOT RECONSTRUCTION SYSTEM

## DEFINITIONS

Symbols and abbreviations may be used on the package label. The following table provides the definition of these symbols and abbreviations.

Table 1. Definitions of Symbols and Abbreviations

Symbol	Definition
	Batch code
	Catalog number
	Do not re-use
	Caution, consult accompanying documents
	Consult operating instructions
	Use by
	Temperature limitation
	Keep dry
	Keep away from sunlight
	Date of manufacture
	Manufacturer

EC REP	Authorized EC Representative in the European Community
STERILEO	Sterilized using ethylene oxide
STERILE R	Sterilized using radiation
STERILE GAS	Sterilized using gas plasma
STERILE A	Sterilized using aseptic processing techniques
R ONLY	For prescription use only
<b>Abbreviation</b>	<b>Material</b>
Ti	Titanium
Ti6Al4V	Titanium Alloy
CoCr	Cobalt Chrome Alloy
SS	Stainless Steel
UHMWPE	Ultra High Molecular Weight Polyethylene

## I. GENERAL PRODUCT INFORMATION

Through the advancement of surgical fusion hardware, the surgeon has been provided a means of correcting deformity and reducing pain for many patients. While the implants used are largely successful in attaining these goals, it must be recognized that they are manufactured from metal, and that no implant can be expected to withstand the activity levels and loads as would normal, healthy bone after fusion occurs.

Each patient must be evaluated by the surgeon to determine the risk/benefit relationship.

In using fusion implants, the surgeon should be aware of the following:

- **The correct selection and sizing of the implant is extremely important.** Selection of the proper size, shape, and design of the implant increases the potential for success. The implants require careful seating and adequate bone support.
- **In selecting patients for surgery, the following factors can be critical to the eventual success of the procedure:**
  1. **Patient's occupation or activity.** If the patient is involved in an occupation or activity which includes substantial lifting or muscle strain, the resultant forces can cause failure of the fixation, the device, or both. The implant will not restore function to the level expected with normal healthy bone, and the patient should not have unrealistic functional expectations.
  2. **Condition of senility, mental illness, or alcoholism.** These conditions, among others, may cause the patient to ignore certain necessary limitations and precautions in the use of the implant, leading to failure or other complications.
  3. **Foreign body sensitivity.** Where material sensitivity is suspected, appropriate tests should be made prior to material selection or implantation.

## **A. PATIENT SELECTION**

Use of surgical fusion hardware requires consideration of the following general indications:

- Good condition of the patient
- Good neurovascular status
- Adequate skin coverage
- Possibility of a functional musculotendinous system
- Adequate bone stock to receive implant
- Availability of post-operative therapy
- Cooperative patient

See Section II for specific product information.

## **B. GENERAL SURGICAL CONTRAINDICATIONS**

- Infection
- Physiologically or psychologically inadequate patient
- Inadequate skin, bone, or neurovascular status
- Irreparable tendon system
- Possibility for conservative treatment
- Growing patients with open epiphyses
- Patients with high levels of activity

### **C. POTENTIAL COMPLICATIONS AND ADVERSE REACTIONS**

In any surgical procedure, the potential for complications exists. The risks and complications with these implants include:

- Infection or painful, swollen or inflamed implant site
- Fracture of the implant
- Loosening or dislocation of the implant requiring revision surgery
- Bone resorption or over-production
- Allergic reaction(s) to implant material(s)
- Untoward histological responses possibly involving macrophages and/or fibroblasts
- Migration of particle wear debris possibly resulting in a bodily response
- Embolism

See Section II for specific product information.

### **D. PRECAUTIONS**

Following the instructions for use provided in product literature can minimize the potential for complications or adverse reactions with any implant.

It is the responsibility of each surgeon using implants to consider the clinical and medical status of each patient and to be knowledgeable about all aspects of implant procedure and the potential complications that may occur. The benefits derived from implant surgery may not meet the patient's expectations or may deteriorate with time, necessitating revision surgery to replace the implant or to carry out alternative procedures. Revision surgeries with implants are common. The patient's mental status



must also be considered. Willingness and/or ability to follow post-operative instructions may also impact the surgical outcome. Surgeons must balance many considerations to achieve the best result in individual patients.

**IF EXCESSIVE LOADING CANNOT BE PREVENTED, AN IMPLANT SHOULD NOT BE USED.**

The main goal of surgery with this implant is to establish bony fusion. Abnormal or excessive forces could lead to delayed union, non-union, or failure of the implant.

Abnormal force loading and subsequent wear may be caused by:

- Uncorrected instability
- Improperly sized implant
- Inadequate soft tissue support
- Implant malposition
- Excessive motion
- Uncorrected or recurrent deformity
- Patient misuse or overactivity

Proper fixation at the time of surgery is critical to the success of the procedure. Bone stock must be adequate to support the device.

Some preventative measures to consider to minimize the potential for complications:

- Follow guidelines for indications and contraindications provided below
- Identify prior pathology

- Stabilize collapse deformities
- Bone graft pre-existing cysts
- Use a properly sized implant
- Avoid K-wires and sutures through the implant
- Avoid open wedge osteotomies of the lower tibia which could be a higher load bearing environment.

Avoid flawing implant surfaces to minimize the potential for early fatigue failure.

If complications develop, possible corrective procedures include:

- Implant removal
- Synovectomy
- Bone grafting of cysts
- Replacement of the implant
- Removal of the implant with fusion of the joint

Clinical results depend on surgeon and technique, pre-operative and post-operative care, the implant, patient pathology and daily activity. It is important that surgeons obtain appropriate informed consent and discuss the potential for complications with each patient prior to surgery. This may include a review of alternative, non-implant procedures such as soft tissue reconstruction or arthrodesis. .

### **Recommendations Regarding Device Fragments**

- Use medical devices in accordance with their labeled indications and Wright Medical Technology's instructions for use, especially during insertion and removal.

- Inspect devices **prior to use** for damage during shipment or storage or any out-of-box defects that might increase the likelihood of fragmentation during a procedure.
- Inspect devices **immediately upon removal from the patient** for any signs of breakage or fragmentation.
- If the device is damaged, retain it to assist with Wright Medical Technology's analysis of the event.
- Carefully consider and discuss with the patient (if possible) the risks and benefits of retrieving vs. leaving the fragment in the patient.
- Advise the patient of the nature and safety of unretrieved device fragments including the following information:
  - a. The material composition of the fragment (if known);
  - b. The size of the fragment (if known);
  - c. The location of the fragment;
  - d. The potential mechanisms for injury, e.g., migration, infection;
  - e. Procedures or treatments that should be avoided such as MRI exams in the case of metallic fragments. This may help to reduce the possibility of a serious injury from the fragment.

## **E. PRECAUTIONS**

Following the instructions for use provided in product literature can minimize the potential for complications or adverse reactions with any implant.

It is the responsibility of each surgeon using implants to consider the clinical and medical status of each patient and to be knowledgeable about all aspects of implant

procedure and the potential complications that may occur. The benefits derived from implant surgery may not meet the patient's expectations or may deteriorate with time, necessitating revision surgery to replace the implant or to carry out alternative procedures. Revision surgeries with implants are common. The patient's mental status must also be considered. Willingness and/or ability to follow post-operative instructions may also impact the surgical outcome. Surgeons must balance many considerations to achieve the best result in individual patients.

**IF EXCESSIVE LOADING CANNOT BE PREVENTED, AN IMPLANT SHOULD NOT BE USED.**

The main goal of surgery with this implant is to establish bony fusion. Abnormal or excessive forces could lead to delayed union, non-union, or failure of the implant.

Abnormal force loading and subsequent wear may be caused by:

- Uncorrected instability
- Improperly sized implant
- Inadequate soft tissue support
- Implant malposition
- Excessive motion
- Uncorrected or recurrent deformity
- Patient misuse or overactivity

Proper fixation at the time of surgery is critical to the success of the procedure. Bone stock must be adequate to support the device.

Some preventative measures to consider to minimize the potential for complications:

- Follow guidelines for indications and contraindications provided above
- Identify prior pathology
- Stabilize collapsed deformities
- Bone graft pre-existing cysts
- Use a properly sized implant
- Avoid K-wires and sutures through the implant
- Avoid open wedge osteotomies of the lower tibia which could be a higher load bearing environment.

Avoid flawing implant surfaces to minimize the potential for early fatigue failure.

If complications develop, possible corrective procedures include:

- Implant removal
- Synovectomy
- Bone grafting of cysts
- Replacement of the implant
- Removal of the implant with fusion of the joint

Clinical results depend on surgeon and technique, pre-operative and post-operative care, the implant, patient pathology and daily activity. It is important that surgeons obtain appropriate informed consent and discuss the potential for complications with each patient prior to surgery. This may include a review of alternative, non-implant procedures such as soft tissue reconstruction or arthrodesis.

### Recommendations Regarding Device Fragments

- Use medical devices in accordance with their labeled indications and Wright Medical Technology's instructions for use, especially during insertion and removal.
- Inspect devices prior to use for damage during shipment or storage or any out-of-box defects that might increase the likelihood of fragmentation during a procedure.
- Inspect devices immediately upon removal from the patient for any signs of breakage or fragmentation.
- If the device is damaged, retain it to assist with Wright Medical Technology's analysis of the event.
- Carefully consider and discuss with the patient (if possible) the risks and benefits of retrieving vs. leaving the fragment in the patient.
- Advise the patient of the nature and safety of unretrieved device fragments including the following information:
  - a. The material composition of the fragment (if known);
  - b. The size of the fragment (if known);
  - c. The location of the fragment;
  - d. The potential mechanisms for injury, e.g., migration, infection;
  - e. Procedures or treatments that should be avoided such as MRI exams in the case of metallic fragments. This may help to reduce the possibility of a serious injury from the fragment.

## **F. MRI SAFETY INFORMATION**

The SALVATION™ Midfoot Reconstruction System implants have not been evaluated for safety and compatibility in the MR environment. These implants have not been tested for heating, migration, or image artifact in the MR environment. The safety of the SALVATION™ Midfoot Reconstruction System in the MR environment is unknown. Scanning a patient who has this device may result in patient injury.

### **MRI Warnings**

There are inherent risks associated with the use of metallic implants in the MR environment: including component migration, heating, and signal interference or image distortion near the component(s). Heating of metallic implants is a risk related to component geometry and material, as well as the MR power, duration, and pulse sequence. Since MR equipment is not standardized, the severity and likelihood of occurrence of these adverse effects are unknown for these implants.

Since these devices have not been tested, Wright cannot make a recommendation for the use of MRIs with these implants, neither for safety considerations nor imaging accuracy.

These components are passive metallic devices. As with all passive devices, there is potential for interference with certain imaging modalities including image distortion for MR and X-ray scatter in CT.

## **G. HANDLING AND STERILIZATION**

### **IMPLANTS**

The implants in this system are either provided sterile or non-sterile; the individual product's labeling will determine whether or not it is packaged sterile. Implants that are

presented in instrument trays are provided non-sterile.

Implants in sterile packaging should be inspected to ensure that the packaging has not been damaged or previously opened. The implants should be opened using aseptic OR technique; they should only be opened after the correct size has been determined.

An implant should never be re-sterilized after contact with body tissues or fluids.

Devices labeled for single-use only should never be reused. Reuse of these devices may potentially result in serious patient harm. Examples of hazards related to the reuse of these devices include, but are not limited to: significant degradation in device performance, cross-infection, and contamination.

Implants provided non-sterile should be processed according to the recommended parameters for instruments (below).

## INSTRUMENTS

Surgical instruments (and non-sterile implants) should be cleaned and sterilized according to the following parameters:

### **CLEANING**

1. **Disassemble** as per manufacturer instructions (if appropriate).
2. **Rinse** with cold tap water to remove gross contamination.
3. **Bathe** in an enzymatic detergent solution prepared per manufacturer directions for 5 minutes.
4. **Scrub** thoroughly with a soft brush and/or pipe cleaner; repeatedly flush any very narrow lumens with enzymatic detergent solution using a syringe.



5. **Rinse** with cold tap water for a minimum of one minute; use a syringe to repeatedly flush any very narrow lumens.
6. **Bathe** in a detergent solution prepared per manufacturer directions for 5 minutes.
7. **Scrub** thoroughly with a soft brush and/or pipe cleaner; repeatedly flush any very narrow lumens with detergent solution using a syringe.
8. **Rinse** thoroughly /flush with deionized / reverse osmosis (RO/DI) water.
9. **Sonicate** for a minimum of 10 minutes in an enzymatic detergent solution prepared per manufacturer directions.
10. **Rinse** thoroughly /flush with RO/DI water.
11. **Dry** with a clean, soft, absorbent, disposable cloth.
12. **Visually inspect** for cleanliness. All visible surfaces, internal and external, should be visually inspected. If necessary re-clean until it is visibly clean.

Steam Sterilization		
Cycle Type	Parameter	Minimum Set Point
270°F (132°C)	Prevacuum	Exposure Temperature
		270°F (132°C)
		Exposure Time
		4 minutes
		Dry Time
		20 minutes

**Note:** Brushes (i.e. pipe cleaners) could be used for cleaning most lumens, however, the use of a syringe to flush narrow lumens with diameters less than or equal to 0.041 inches is recommended.

### **Sterilization**

1. Double wrap the component in an FDA-cleared CSR wrap or a similar type non-woven medical grade wrapping material.
2. Autoclave according to the following parameters:
3. After sterilization, remove the component from its wrapping using accepted sterile technique with powder-free gloves. Ensure that implants are at room temperature prior to implantation. Avoid contact with hard objects that may cause damage.

These recommended instructions are consistent with AAMI ST79 Table 5 guidelines and have been developed and tested using specific equipment. The recommended instructions provided above have been validated by Wright Medical. It remains the responsibility of the processor to ensure that the processing is performed using appropriate equipment and materials and achieves the desired result. All users should be qualified personnel and properly trained on applicable policies, procedures, and standards.

For additional information see Wright's "Cleaning and Handling of Wright Medical Instruments".

### **G. STORAGE CONDITIONS**

All implants must be stored in a clean, dry environment and be protected from sunlight and extremes in temperature.

## II. SPECIFIC PRODUCT INFORMATION

### A. SALVATION™ MIDFOOT RECONSTRUCTION SYSTEM

#### DESCRIPTION

The SALVATION™ Midfoot Reconstruction System, which is comprised of Bolts, Beams, and Nails, was designed to address the demands of advanced midfoot reconstruction. The 5.0mm Fusion Bolt and 6.5mm Fusion Bolt feature a solid core design. The 7.0 Fusion Beam features a cannulated design to facilitate placing the implant over a K-Wire. The 8.0mm Nail features a compression slot for internal compression with a cross screw for rotational stability. All implants are made from Ti6Al4V (titanium alloy).

#### INDICATIONS

The SALVATION™ Midfoot Reconstruction System is indicated for fracture fixation, osteotomies, reconstruction procedures, non-unions, and fusions of bones in the foot and ankle including the metatarsals, cuneiforms, cuboid, navicular, calcaneus and talus; specific examples include: intramedullary medial column fusion and lateral column fusion resulting from neuropathic osteoarthropathy (Charcot).

**CAUTION:** Federal Law (U.S.) restricts this device to sale or use by or on the order of a physician. ™ and ® denote Trademarks and Registered Trademarks of Wright Medical Group N.V. or its affiliates.

**Left**

**Blank**

**Intentionally**

**Left**

**Blank**

**Intentionally**

**Left**

**Blank**

**Intentionally**

**Left**

**Blank**

**Intentionally**

**Left**

**Blank**

**Intentionally**



**Left**

**Blank**

**Intentionally**

**Left**

**Blank**

**Intentionally**